

Cushion Rigidizer

Installation and Operating Instructions

NOTE: Check all parts for shipping damages before using. In case of damage, do NOT use the equipment. Contact the Equipment Supplier for further instructions.

WARNING

DO NOT install this equipment without first reading and understanding this instruction booklet. If you are unable to understand these instructions, contact a healthcare professional, dealer or technical personnel before attempting to install this equipment - otherwise, injury or damage may occur.

NOTICE

Information contained within this document is subject to change without notice.

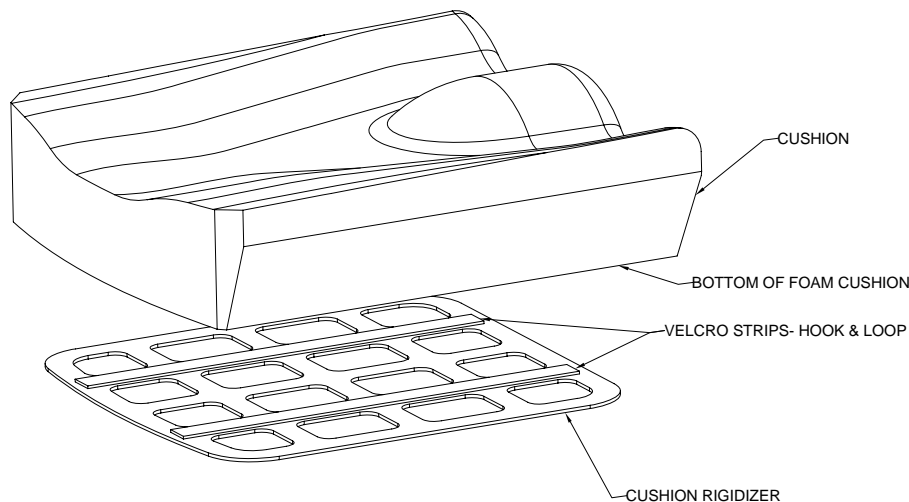


Fig.1 (cover not shown)

Installing Cushion Rigidizer /Fig.1/

1. Remove seat cushion from chair.
2. Open seat cover and remove the foam cushion from seat cover.

Note: Do not remove the backing from Velcro loop strip Tape until you have marked the placement area on the foam cushion.

3. Position Cushion Rigidizer onto the bottom of the foam cushion to the extreme rear while centering it on the left and right sides.
4. Outline the foam cushion with marker or pen using the Cushion Rigidizer as template to mark the placement area.

5. After marking placement area, remove the Velcro backing from Cushion Rigidizer and install Rigidizer onto bottom of foam cushion. Seat Rigidizer installation and operating Manual
6. Press down firmly to adhere the Cushion Rigidizer to the foam cushion.
7. Re- install foam cushion into the seat cover and install cushion onto the chair.

Installation warnings

Ensure that Cushion Rigidizer is properly installed per instruction.

Do not use the Cushion Rigidizer on wheelchairs equipped with rigid seat pan. Use only on sling wheelchairs.

The Cushion Rigidizer does not change the weight limitation of your wheelchair

Cushion Rigidizer Inspection

1. Once a week, visually inspect all parts for deformation, corrosion, breakage, wear and/or compression.
2. Do not continue to use this product if any problems are discovered. Corrective maintenance can be performed at or arranged through your equipment supplier

WARRANTY

This warranty is extended only to the original purchaser/user of our products.

Future Mobility Healthcare Inc. warrants this seating product to be free from defects in materials and workmanship for one (1) year on cushions and 90 days on covers upon normal usage by original purchaser. If within this warranty period the product shall be proven to be defective, such product shall be repaired or replaced, at Future Mobility Healthcare Inc. discretion. Future Mobility Healthcare Inc. sole obligation and your exclusive remedy under this warranty shall be limited to the repair and/or replacement of the product or its parts. This warranty does not include any labour or shipping charges incurred in replacement part installation or repair of any product.

For warranty service, please contact the dealer from whom you purchased your Future Healthcare Inc. product. In the event you do not receive satisfactory warranty service, please write directly to Future Mobility Healthcare Inc. at 3223 Orlando Drive, Mississauga, Ontario, L4V 1C5. Provide the dealer's name, address, model number, date of purchase and indicate the nature of the defect.

DO NOT return products to Future Mobility Healthcare Inc. our prior consent. The defective unit or parts must be returned for warranty inspection within thirty (30) days of the return authorization date. (Future Mobility Healthcare Inc. will issue a return authorization number). Please prepay all shipping charges; C.O.D. shipments will be refused.

LIMITATIONS and EXCLUSIONS: This warranty shall not apply to problems arising from normal wear or failure to adhere to the enclosed instructions. Products subjected to negligence, accident, improper usage, maintenance or storage; or products modified without Future Mobility Healthcare Inc. written consent including, but not limited to : modification through the use of any unauthorized parts or attachments; products damaged by reason or repairs made to any component without the specific consent of Future Mobility Healthcare Inc., or products repaired by anyone other than a Future Mobility Healthcare Inc. dealer. Such evaluation shall be determined by Future Mobility Healthcare Inc..

The foregoing warranty is exclusive and in lieu of all other expressed warranties. It shall not extend beyond the duration of the expressed warranty provided herein and the remedy for violations of any implied warranty shall be limited to repair or replacement of the defective product pursuant to the terms contained herein. Future Mobility Healthcare Inc. shall not be liable for any consequential or incidental damages whatsoever.

This warranty shall be extended to comply with all provincial laws and requirements.

3223 Orlando Drive
Mississauga ON L4V 1C5
Tel: 905-671-1661
Fax: 905-671-3377

e-mail: customersupport@future-mobility.com
www.future-mobility.com

**Future Mobility
Healthcare Inc.**