

FUTURE
MOBILITY HEALTHCARE INC.

HEADREST

PARTS MANUAL

FUTURE MOBILITY HEALTHCARE APPRECIATES YOUR BUSINESS.
OUR CUSTOMER SERVICE REPRESENTATIVES CAN BE CONTACTED AT:

**TO PLACE ORDERS:
(905) 671-1661**

FOR YOUR CONVENIENCE YOU MAY ALSO PLACE ORDERS BY FAX AT:

FAX: (905) 671-3377

PLEASE VISIT OUR WEBSITE AT:

www.futuremobility.ca

*Part prices and configurations included in this Parts Manual are subject to change without prior notice.


* All prices are shown as per unit individual price.

CONTENTS

HEADREST VERTICAL TUBES.....	5
HEADREST FIXED VERTICAL TUBE.....	5
HEADREST MULTI-AXIAL VERTICAL TUBE	7
HEADREST INFINITE VERTICAL TUBE	9
SQ HEADREST MULTI-AXIAL VERTICAL TUBE	11
HEADREST HORIZONTAL TUBES	13
HEADREST FIXED HORIZONTAL TUBE	13
HEADREST ARTICULATED HORIZONTAL TUBE	13
HEADREST INFINITE HORIZONTAL TUBE	13
STAINLESS COMMODOE ADJUSTABLE HEADREST	15
ADJUSTABLE HEADREST ASSY	15
LINKS HEADREST.....	17
LINKS STRAIGHT MASTER ASSY	17
LINKS CURVED MASTER ASSY	19
LINKS HEADREST STRAIGHT VERTICAL TUBE	21
LINKS HEADREST CURVED VERTICAL TUBE	23
LINKS HEADREST BALL JOINT	25
LINKS HEADREST SINGLE LINK.....	25
HEADREST BASE ADAPTER BRACKET	27
2-HOLE HEADREST BASE ADAPTER BRACKET	27
UNIVERSAL HEADREST BASE ADAPTOR BRACKET	29
HEADREST CURVED HINGED OVAL CLAMP.....	31
HEADREST CIRCULAR HINGED OVAL CLAMP.....	33

HEADREST VERTICAL TUBES

HEADREST FIXED VERTICAL TUBE

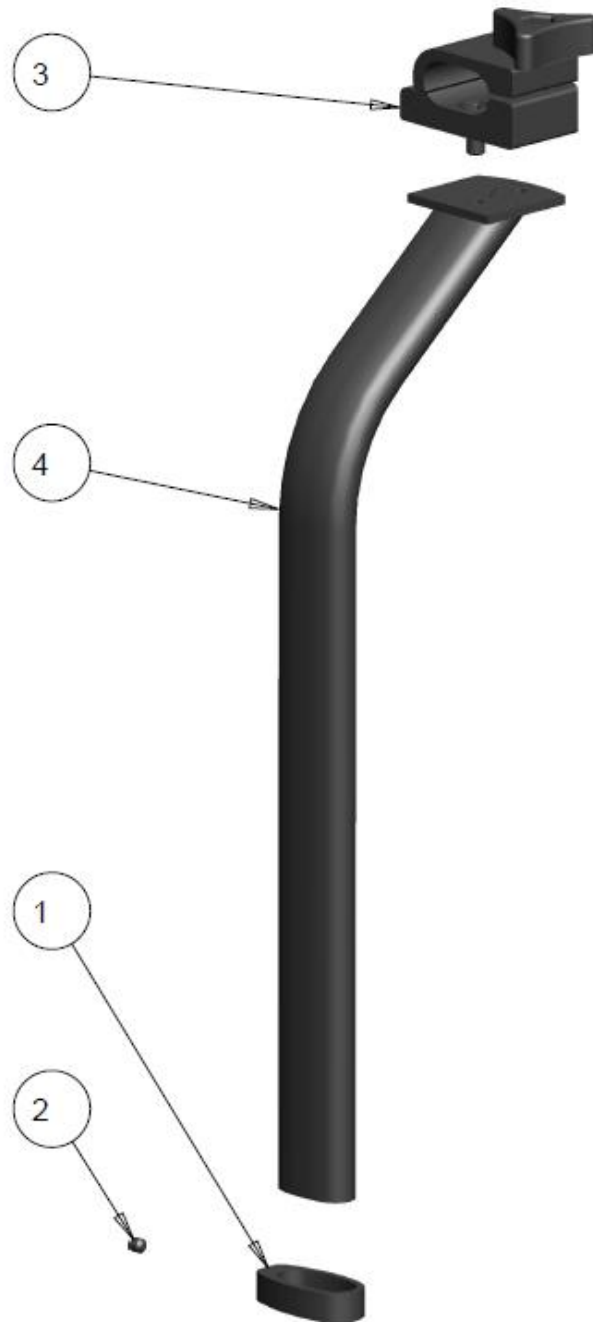


HEADREST FIXED VERTICAL TUBE TABLE

ITEM	QTY	NOTES	PART NO.	DESCRIPTION	PRICE/ PER UNIT
1	1		1001199	HEADREST LOCKING COLLAR	\$17.50
2	1		1001214	SCREW, SET, SOC HD CUP POINT #10-32 X 3/16	\$3.00
3	1		1019596	CURVED HINGED OVAL HEADREST CLAMP	\$48.00
4	1		1001393	HEADREST CURVED POST WELDMENT	\$52.50
	1		1003928	HEADREST CURVED POST WELDMENT + 4"	\$72.00
	1		1018183	HEADREST STRAIGHT POST WELDMENT	\$47.00
	1		1021424	KIT, CURVED POST STANDARD (ITEMS 1-4)	\$110.00
	1		1021425	KIT, CURVED POST PLUS 4" (ITEMS 1-4)	\$135.00
	1		1021426	KIT, STRAIGHT POST STANDARD (ITEMS 1-4)	\$105.00

NOTES:

HEADREST MULTI-AXIAL VERTICAL TUBE

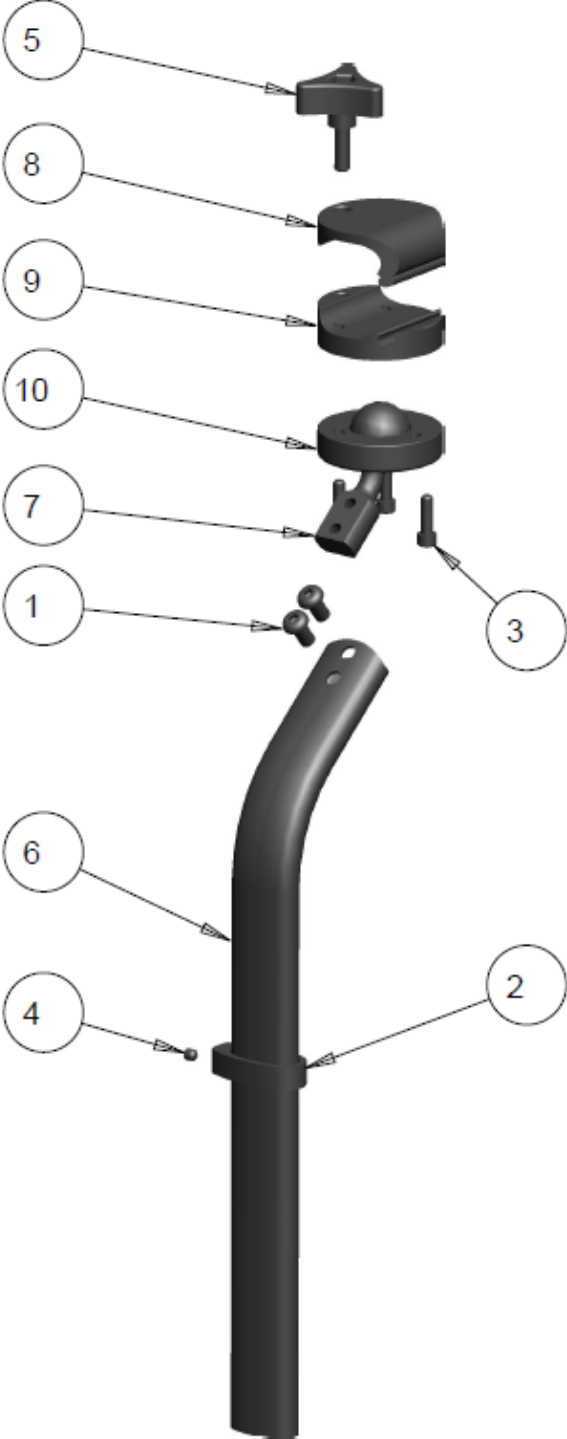

HEADREST MULTI-AXIAL VERTICAL TUBE TABLE

ITEM	QTY	NOTES	PART NO.	DESCRIPTION	PRICE/ PER UNIT
1	2		1000411	SCREW,SOC BUTTON HEAD 1/4-20 X 0.500	\$3.00
2	1		1001199	HEADREST LOCKING COLLAR	\$17.50
3	3		1001203	SCREW, SOCKET HEAD CAP, 10-32NF X 5/8	\$3.25
4	1		1001214	SCREW, SET, SOC HD CUP POINT #10-32 X 3/16	\$3.00
5	1		1001464	HANDLE 3-POINT 1/4-20 X 3/4 STEM BLACK	\$8.00
6	1		1005791	DUAL-MULTIAXIAL HEADREST VERT POST WELDMENT	\$52.00
	1		1007004	STANDARD DUAL- MULTIAXIAL HEADREST VERT POST WELDMENT	\$55.00
	1		1007005	+4" DUAL- MULTIAXIAL HEADREST VERT POST WELDMENT	\$72.00
	1		1007006	+6 " DUAL- MULTIAXIAL HEADREST VERT POST WELDMENT	\$78.00
7	1		1007001	BALL-JOINT, 1" DIA CURVED	\$35.00
8	1		1020988	HINGED CIRCULAR UPPER CLAMP	\$32.00
9	1		1032651	HINGED CIRCULAR LOWER CLAMP	\$38.00
10	1		1032847	FACE PLATE, BALL SOCKET CLAMP HEAVY	\$14.00
	1		1033159	KIT,DUAL-MULTIAXIAL HEADREST VERT POST WELDMENT	\$190.25
	1		1033160	KIT, DUAL-MULTIAXIAL VERTICAL, HEADREST WLDMNT-STD.	\$200.55
	1		1033161	KIT, DUAL-MULTIAXIAL VERTICAL, HEADREST WLDMNT +4"	\$210.85
	1		1033162	KIT, DUAL-MULTIAXIAL VERTICAL, HEADREST WLDMNT +6"	\$220.25

NOTES:

HEADREST INFINITE VERTICAL TUBE




HEADREST INFINITE VERTICAL TUBE TABLE

ITEM	QTY	NOTES	PART NO.	DESCRIPTION	PRICE/ PER UNIT
1	2		1000411	SCREW,SOC BUTTON HEAD 1/4-20 X 0.500	\$3.00
2	1		1001199	HEADREST LOCKING COLLAR	\$17.50
3	1		1001214	SCREW, SET, SOC HD CUP POINT #10-32 X 3/16	\$3.00
4	1		1001464	HANDLE 3-POINT 1/4-20 X 3/4 STEM BLACK	\$8.00
5	1		1005791	GEN DUAL-MULTIAXIAL HEADREST VERT POST WELDMENT	\$52.00
	1		1007005	MULTIAXIAL HEADREST CURVED VERT POST + 4 WELDMENT	\$72.00
	1		1020201	MULTIAXIAL HEADREST STRAIGHT VERT POST WELDMENT	\$47.00
6	1		1007001	BALL-JOINT, 1" DIA CURVED	\$35.00
7	1		1016938	SINGLE HANDLE MAIN HOUSING RT	\$57.00
8	1		1016939	SINGLE HANDLE MAIN HOUSING LT	\$57.00
	1		1016937	KIT, INFINITE CURVED POST STANDARD (ITEMS 1-8)	\$215.00
	1		1021428	KIT, INFINITE CURVED POST + 4" (ITEMS 1-8)	\$235.00
	1		1020999	KIT, INFINITE STRAIGHT POST (ITEMS 1-8)	\$210.00

NOTES:

SQ HEADREST MULTI-AXIAL VERTICAL TUBE

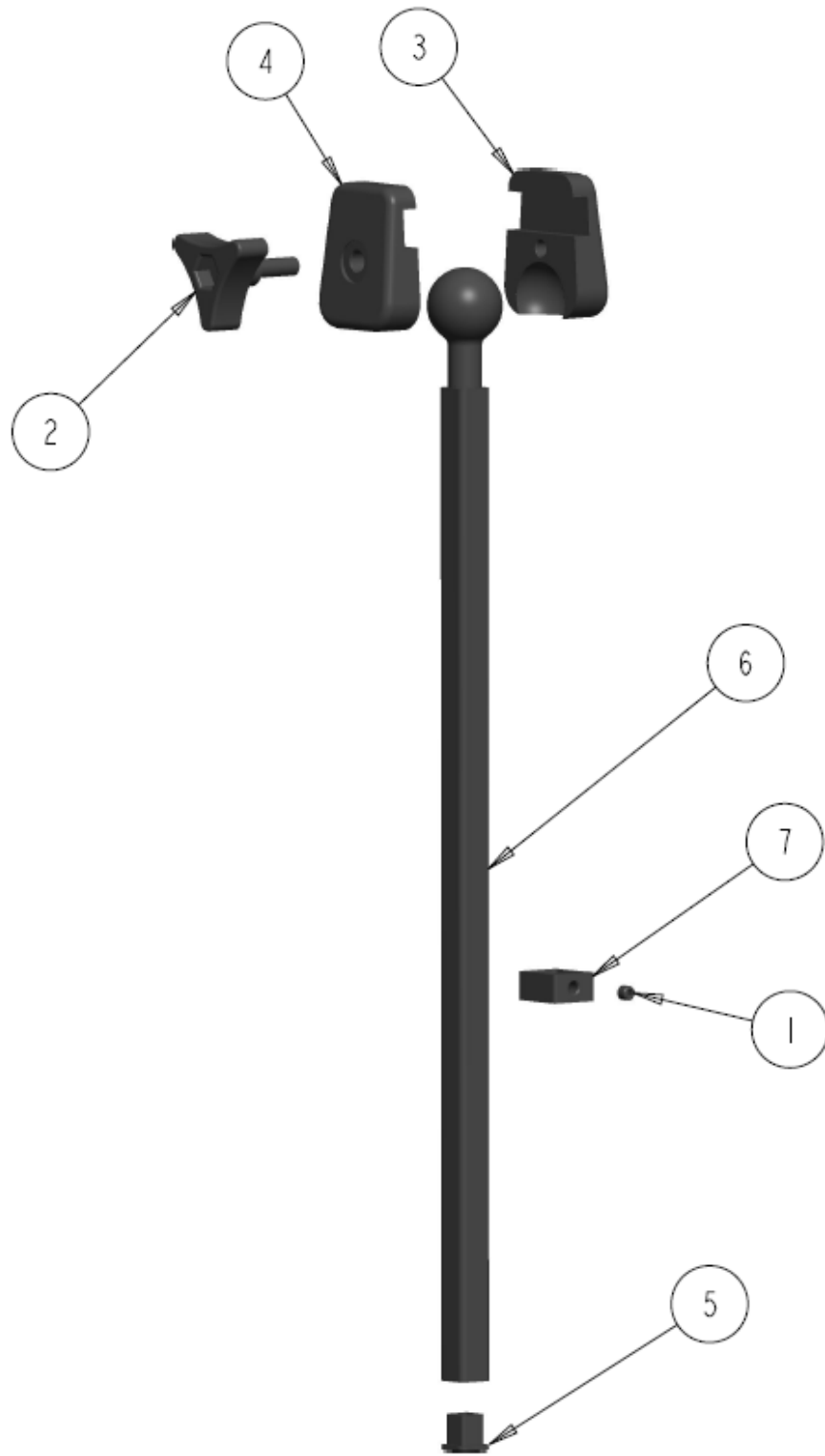


SQ HEADREST MULTI-AXIAL VERTICAL TUBE - TABLE

ITEM	QTY	NOTES	PART NO.	DESCRIPTION	PRICE/ PER UNIT
1	1		1001214	SCREW, SET, SOC HD CUP POINT #10-32 X 3/16	\$3.00
2	1		1001464	HANDLE 3-POINT 1/4-20 X 3/4 STEM BLACK	\$8.00
3	1		1016938	SINGLE HANDLE MAIN HOUSING RT	\$57.00
4	1		1016939	SINGLE HANDLE MAIN HOUSING LT	\$57.00
5	1		1016945	SQUARE ENDPLUG 0.5 X 0.5	\$2.50
6	1		1033770	VERTICLE SQ TUBE WELD STANDARD	\$72.00
7	1		1033775	SQ POSITIONING COLLAR MACHINED	\$20.50
	1		1037665	KIT, SQ HEADREST MULTI-AXIAL VERTICAL TUBE ASSY RT	\$150.00
			1037666	KIT, SQ HEADREST MULTI-AXIAL VERTICAL TUBE ASSY LT	\$150.00


NOTES:

HEADREST HORIZONTAL TUBES


HEADREST FIXED HORIZONTAL TUBE

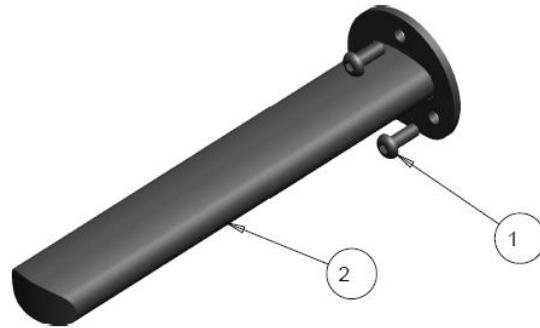

HEADREST ARTICULATED HORIZONTAL TUBE

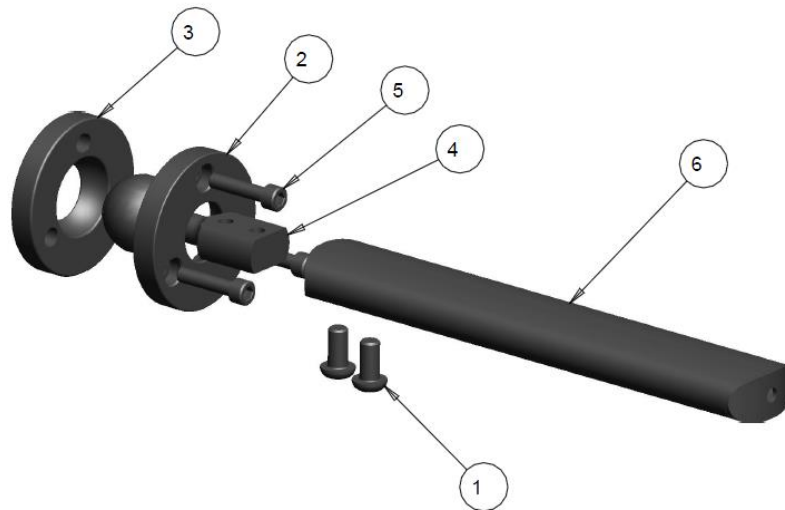

HEADREST INFINITE HORIZONTAL TUBE

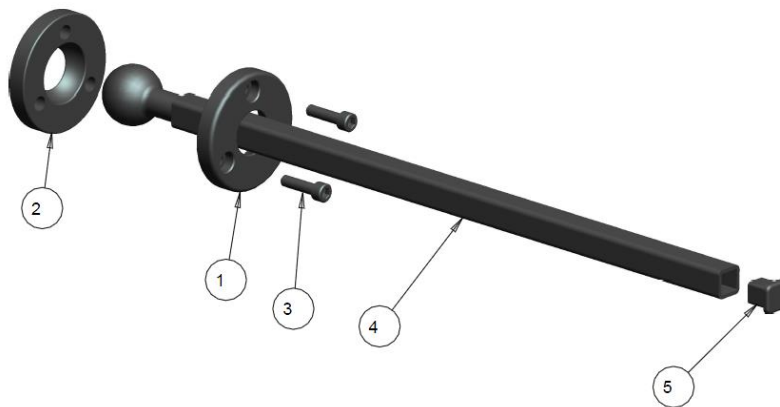

HEADREST FIXED HORIZONTAL TUBE TABLE

ITEM	QTY	NOTES	PART NO.	DESCRIPTION	PRICE/ PER UNIT
1	2		1000411	SCREW,SOC BUTTON HEAD 1/4-20 X 0.500	\$3.00
2	1		1000790	HORIZONTAL FIXED HEADREST WELDMENT	\$49.00
	1		1003047	HORIZONTAL FIXED HEADREST WELDMENT +4	\$59.00
	1		1001208	KIT, HORIZONTAL FIXED HEADREST (ITEMS 1-2)	\$52.00
	1		1004365	KIT, HORIZONTAL FIXED HEADREST +4" (ITEMS 1-2)	\$62.00

NOTES:

HEADREST ARTICULATED HORIZONTAL TUBE TABLE

ITEM	QTY	NOTES	PART NO.	DESCRIPTION	PRICE/ PER UNIT
1	2		1000411	SCREW, SOC BUTTON HEAD 1/4-20 X 0.500	\$3.00
2	1		1001198	FACE PLATE, BALL SOCKET CLAMP	\$20.10
3	1		1001200	SPACER PLATE, BALL SOCKET CLAMP	\$26.32
4	1		1002533	BALL-JOINT, 1" DIA	\$28.00
5	3		1032587	SCREW, SOCKET HEAD CAP,10-32NFX 7/8	\$4.00
6	1		1001210	HORIZONTAL, BALL-JOINT TUBE, HEADREST WLDT STD	\$55.00
	1		1005113	HEADREST BALL-JOINT MOUNT WLDMT+2" LONGER	\$60.00
	1		1002675	HEADREST BALL-JOINT MOUNT WLDMT+3" LONGER	\$65.00
	1		1003980	HEADREST BALL-JOINT MOUNT WLDMT+4" LONGER	\$55.00
	1		1005108	HEADREST BALL-JOINT MOUNT WLDMT+6" LONGER	\$75.00
	1		1032294	HEADREST BALL-JOINT MOUNT WLDMT+7" LONGER	\$80.00
	1		1003427	KIT, HORIZONTAL, ARTICULATED, HEADREST ASS'Y 7" STD	\$90.00
	1		1005118	KIT, HORIZONTAL, ARTICULATED, HEADREST ASS'Y +2" LONGER	\$145.00
	1		1003665	KIT, HORIZONTAL, ARTICULATED, HEADREST ASS'Y +3" LONGER	\$155.00
	1		1003996	KIT, HORIZONTAL, ARTICULATED, HEADREST ASS'Y+ 4" LONGER	\$100.00
	1		1005120	KIT, HORIZONTAL, ARTICULATED, HEADREST ASS'Y+6" LONGER	\$175.00

NOTES:

HEADREST INFINITE HORIZONTAL TUBE TABLE


ITEM	QTY	NOTES	PART NO.	DESCRIPTION	PRICE/ PER UNIT
1	1		1001198	FACE PLATE, BALL SOCKET CLAMP	\$20.10
2	1		1001200	SPACER PLATE, BALL SOCKET CLAMP	\$26.32
3	3		1006241	SCREW, SOCKET HEAD CAP,10-32NFX 3/4	\$3.00
4	1		1016941	HORIZONTAL SQ TUBE WELDMENT (+4")	\$68.00
	1		1021420	HORIZONTAL SQ TUBE WELDMENT	\$63.00
5	1		1016945	SQUARE ENDPLUG 0.5 X 0.5	\$2.50
	1	A	1033163	KIT, HORIZONTAL HEADREST SQ TUBE (+4") (ITEMS 1-5)	\$121.75
	1	A	1034527	KIT, HORIZONTAL HEADREST SQ TUBE (ITEMS 1-5)	\$121.75

NOTES:

A- ONLY TO BE USED WITH INFINITE VERTICAL TUBE HARDWARE

STAINLESS COMMODE ADJUSTABLE HEADREST

ADJUSTABLE HEADREST ASSY

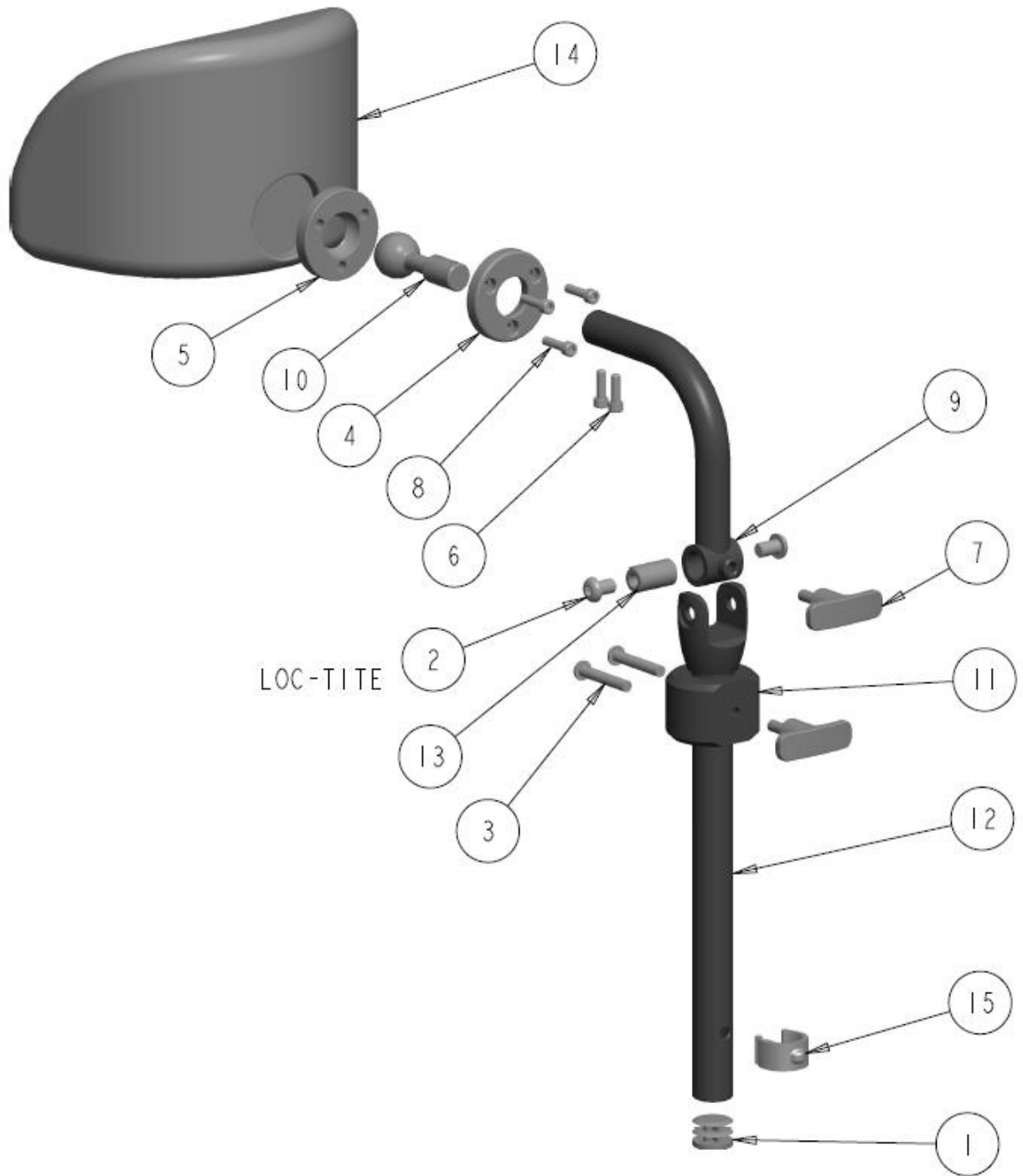

ADJUSTABLE HEADREST ASSY - TABLE

ITEM	QTY	NOTES	PART NO.	DESCRIPTION	PRICE/ PER UNIT
1	1		1000421	PLUG, UNIVERSAL BLACK 0.875 ROUND	\$2.50
2	2		1000413	SCREW, SOCKET HEAD BUTTON 5/16-18 X 0.5	\$3.75
3	2		1000493	SCREW, SOCKET HEAD BUTTON 1/4-20 X 1.50	\$3.00
4	1		1001198	FACE PLATE, BALL SOCKET CLAMP	\$20.10
5	1		1001200	SPACER PLATE, BALL SOCKET CLAMP	\$26.32
6	2		1001203	SCREW, SOCKET HEAD CAP, 10-32NFX 5/8	\$3.25
7	2		1005259	T-HANDLE ROUNDED	\$17.00
8	3		1006241	SCREW, SOCKET HEAD CAP,10-32NFX 3/4	\$3.00
9	1		1011895	HEADREST CURVED STEM WELDMENT	\$13.30
10	1		1011897	SS BALL-JOINT, 1" DIA	\$5.30
11	1		1011898	MAIN LOCK COLLAR HOUSING	\$60.00
12	1		1011900	HEADREST STRAIGHT STEM WELDMENT	\$15.00
13	1		1011903	PIVOT BUSHING, UPPER CLAMP	\$4.00
14	1		1011906	FOAM SKINNED HEADREST HJ-03	\$150.00
15	1		1011908	SNAP CLIP .875 DIA	\$8.50
	1		1037664	KIT, COMPLETE ADJUSTABLE HEADREST ASSY	\$330.00

NOTES :

LINKS HEADREST

LINKS STRAIGHT MASTER ASSY




LINKS STRAIGHT MASTER ASSY – TABLE

ITEM	QTY	NOTES	PART NO.	DESCRIPTION	PRICE/ PER UNIT
1	1	B	REF	HEADREST BASE, UNIVERSAL	REF
2	2	A	REF	STRAIGHT LINK ASSY	REF
3	1	A	REF	LINKS BALL JOINT ASSY	REF
4	1	A	REF	VERTICLE TUBE HANDLE COBRA ASSY	REF

NOTES:

- A. PARTS BREAKDOWN FOUND WITHIN LINKS HEADREST SECTION
- B. PARTS BREAKDOWN FOUND IN HEADREST BASE ADAPTER SECTION

LINKS CURVED MASTER ASSY

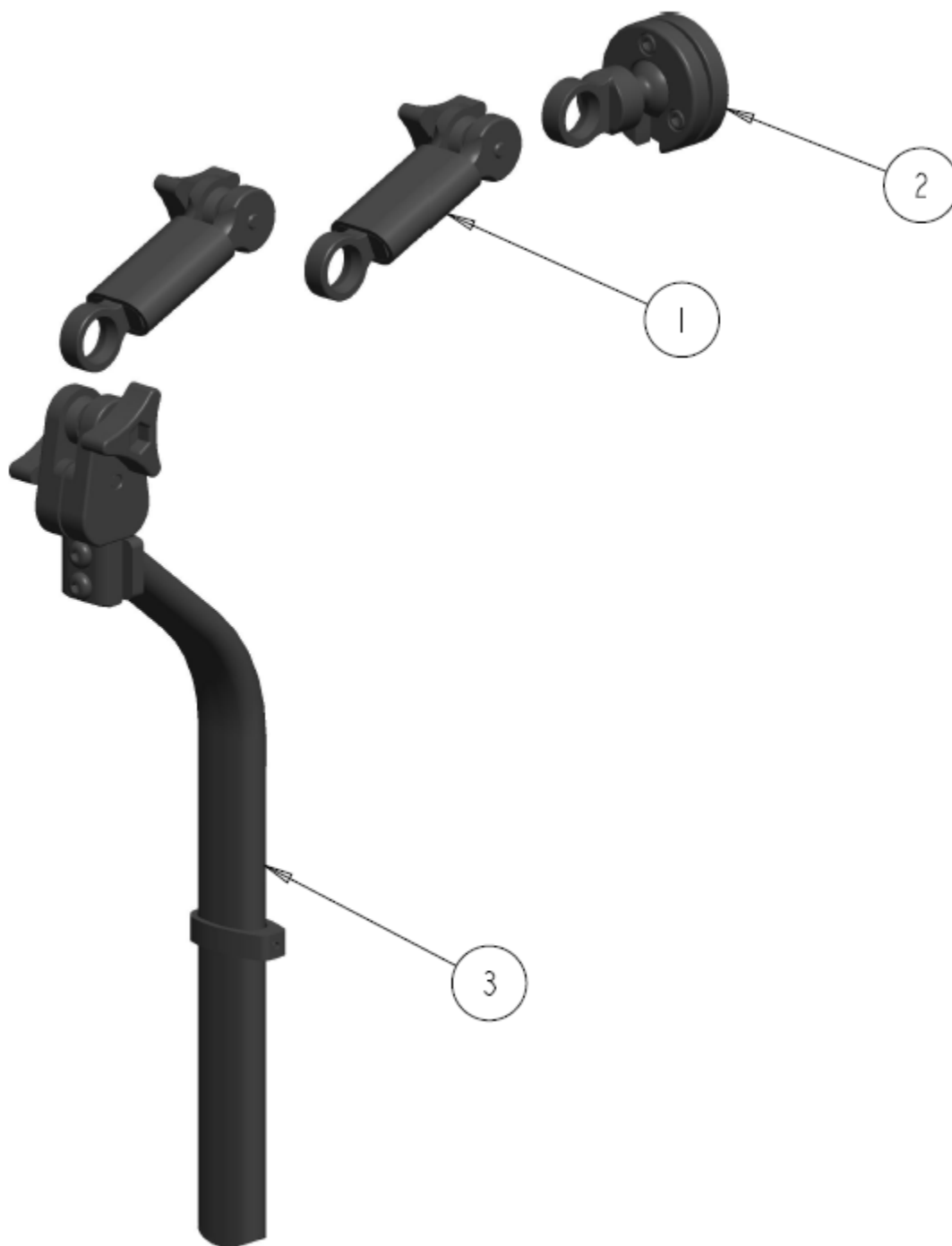


LINKS CURVED MASTER ASSY – TABLE

ITEM	QTY	NOTES	PART NO.	DESCRIPTION	PRICE/ PER UNIT
1	2	A	REF	STRAIGHT LINK ASSY	REF
2	1	A	REF	LINKS BALL JOINT ASSY	REF
3	1	A	REF	COBRA CURVED VERTICLE TUBE	REF

NOTES:

A. PARTS BREAKDOWN FOUND WITHIN LINKS HEADREST SECTION

LINKS HEADREST STRAIGHT VERTICAL TUBE

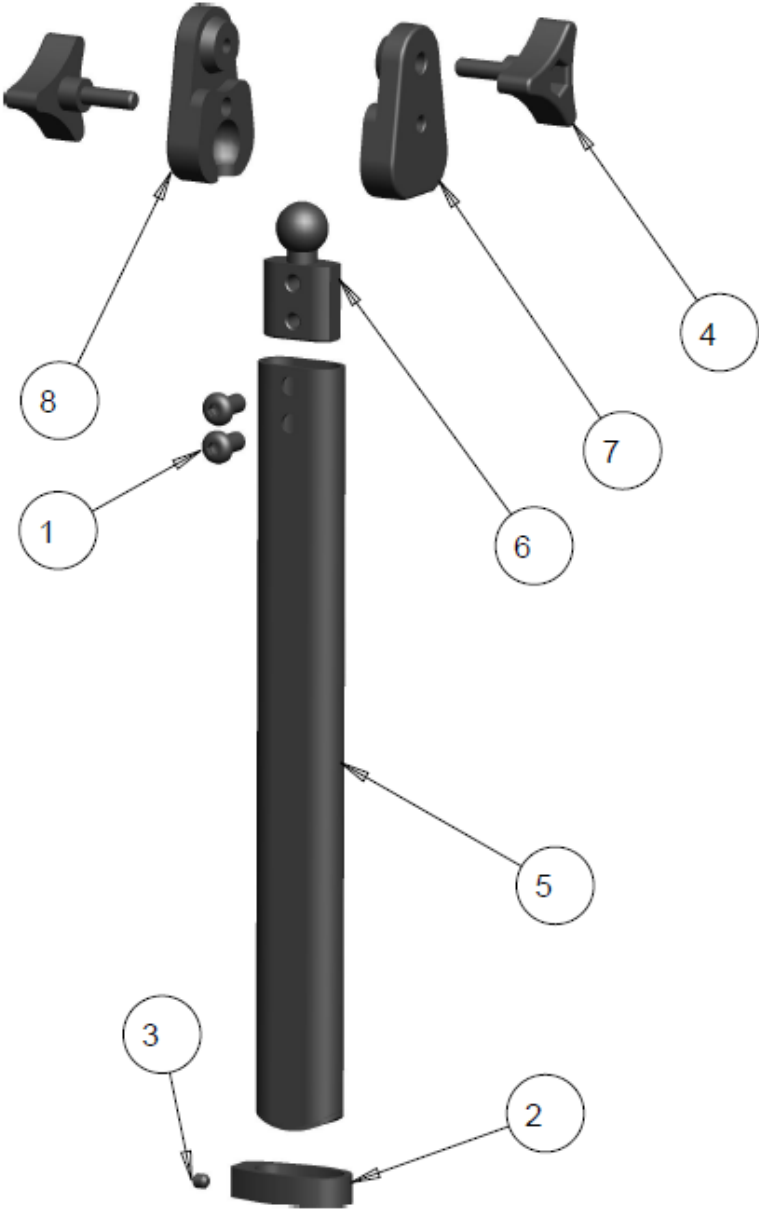

LINKS HEADREST STRAIGHT VERTICAL TUBE TABLE

ITEM	QTY	NOTES	PART NO.	DESCRIPTION	PRICE/ PER UNIT
1	2		1000411	SCREW,SOC BUTTON HEAD 1/4-20 X 0.500	\$3.00
2	1		1001199	HEADREST LOCKING COLLAR	\$17.50
3	1		1001214	SCREW, SET, SOC HD CUP POINT #10-32 X 3/16	\$3.00
4	2		1001464	HANDLE 3-POINT 1/4-20 X 3/4 STEM BLACK	\$8.00
5	1		1014048	VERTICAL OVAL TUBE LINKS WELD	\$44.00
6	1		1014268	BALL-JOINT, .687 DIA	\$32.00
7	1		1015362	BALL JOINT HANDLE MAIN HOUSING RT	\$58.00
8	1		1015363	BALL JOINT HANDLE MAIN HOUSING LT	\$58.00
	1		1034522	KIT, LINKS STRAIGHT POST (ITEMS 1-8)	\$196.00

NOTES:

LINKS HEADREST CURVED VERTICAL TUBE




LINKS HEADREST CURVED VERTICAL TUBE TABLE


ITEM	QTY	NOTES	PART NO.	DESCRIPTION	PRICE/ PER UNIT
1	1		1001199	HEADREST LOCKING COLLAR	\$17.50
2	1		1001214	SCREW, SET, SOC HD CUP POINT #10-32 X 3/16	\$3.00
3	2		1001381	SOC HD SCREW 1/4-20 X 1.00	\$3.00
4	2		1001464	HANDLE 3-POINT 1/4-20 X 3/4 STEM BLACK	\$8.00
5	1		1015362	BALL JOINT HANDLE MAIN HOUSING RT	\$58.00
6	1		1015363	BALL JOINT HANDLE MAIN HOUSING LT	\$58.00
7	1		1015445	CURVED VERTICAL OVAL TUBE COBRA	\$63.00
8	1		1015448	BALL-JOINT, .687 DIA (NOT THREADED)	\$32.00
	1		1034523	KIT, LINKS CURVED POST (ITEMS 1-8)	\$206.00

NOTES:

LINKS HEADREST BALL JOINT

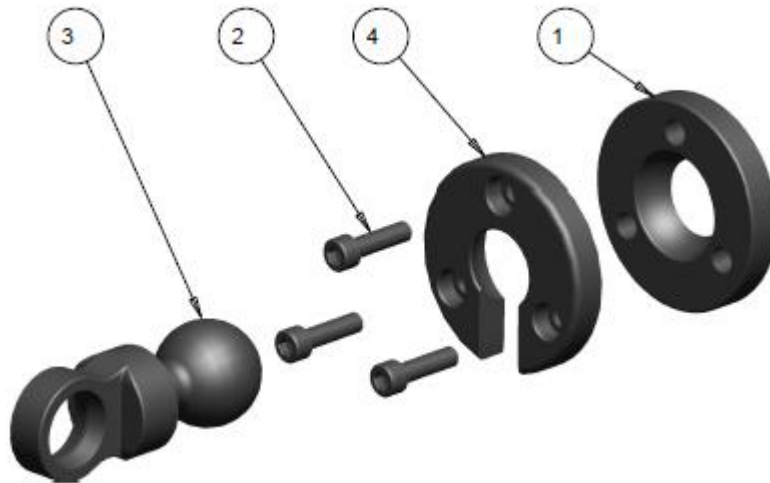

LINKS HEADREST SINGLE LINK

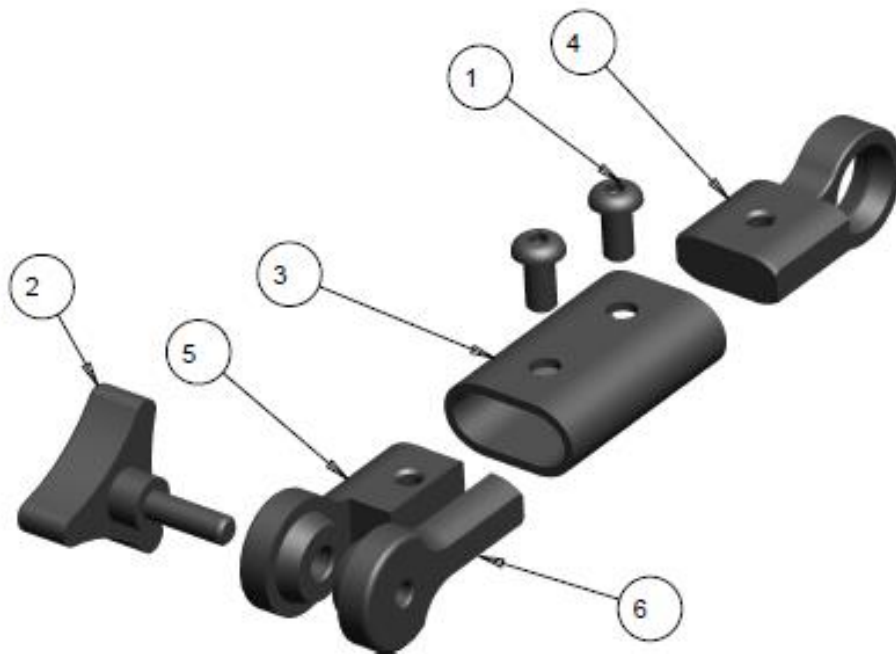

LINKS HEADREST BALL JOINT TABLE

ITEM	QTY	NOTES	PART NO.	DESCRIPTION	PRICE/ PER UNIT
1	1		1001200	SPACER PLATE, BALL SOCKET CLAMP	\$26.32
2	3		1006241	SCREW, SOCKET HEAD CAP,10-32NFX 3/4	\$3.00
3	1		1014042	COBRA BALL JOINT MACHINED	\$25.00
4	1		1032845	FACE PLATE, BALL CLAMP - SLOTTED	\$14.00
	1		1014044	KIT, LINKS BALL JOINT (ITEMS 1-4)	\$44.00

NOTES:


LINKS HEADREST SINGLE LINK TABLE

ITEM	QTY	NOTES	PART NO.	DESCRIPTION	PRICE/ PER UNIT
1	2		1000411	SCREW,SOC BUTTON HEAD 1/4-20 X 0.500	\$3.00
2	1		1001464	HANDLE 3-POINT 1/4-20 X 3/4 STEM BLACK	\$8.00
3	1		1014032	MAIN OVAL TUBE HOUSING	\$16.00
4	1		1014033	RING STRAIGHT LINK MACHINED	\$18.00
5	1		1014034	CONE STRAIGHT LINK MAIN	\$14.00
6	1		1014035	CONE STRAIGHT LINK INSERT	\$10.00
	1		1014031	KIT, ADDITIONAL LINK SINGLE (ITEMS 1-6)	\$50.00

NOTES:

HEADREST BASE ADAPTER BRACKET

2-HOLE HEADREST BASE ADAPTER BRACKET

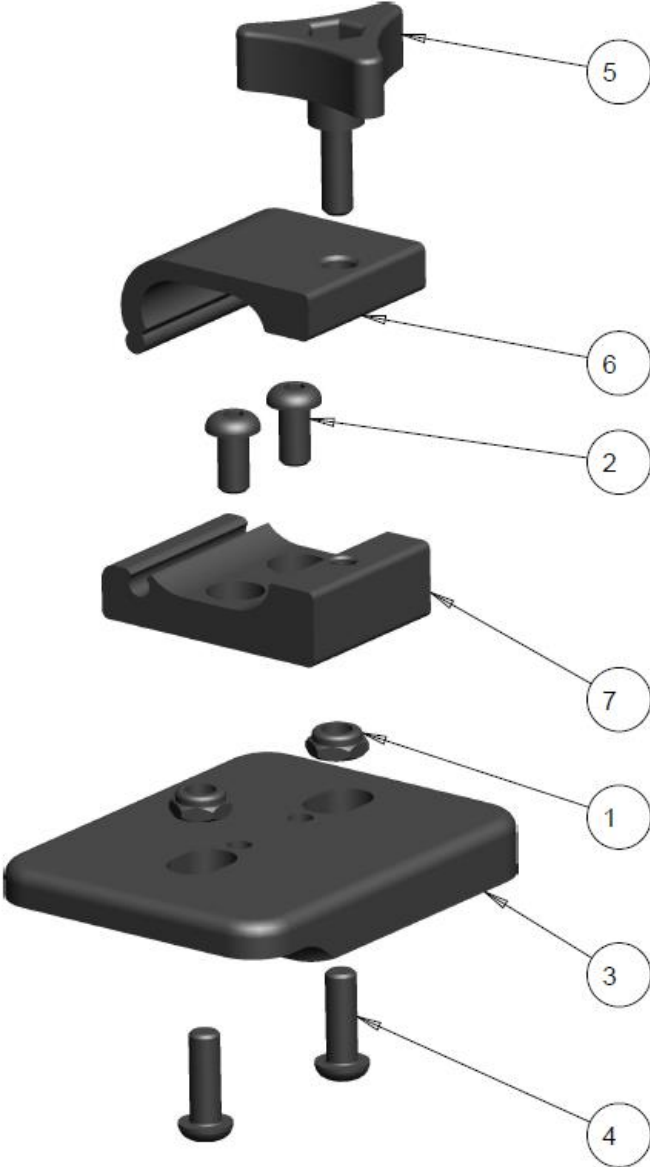


2-HOLE HEADREST BASE ADAPTOR BRACKET TABLE

ITEM	QTY	NOTES	PART NO.	DESCRIPTION	PRICE/ PER UNIT
1	2		1000397	NUT, HEX-NYLOCK THIN, 1/4-20NC	\$2.00
2	2		1000411	SCREW,SOC BUTTON HEAD 1/4-20 X 0.500	\$3.00
3	1		1001367	2-HOLE MOUNTING PLATE	\$50.00
4	2		1001391	SCREW, SOCKET HEAD BUTTON, 1/4-20NC X 3/4	\$3.00
5	1		1001464	HANDLE 3-POINT 1/4-20 X 3/4 STEM BLACK	\$8.00
6	1		1019474	OVAL UPPER CLAMP	\$16.00
7	1		1019475	OVAL TUBE LOWER CLAMP	\$20.00
	1		1034525	KIT, 2-HOLE HEADREST ADAPTER BRACKET (ITEMS 1-7)	\$105.00

NOTES:

UNIVERSAL HEADREST BASE ADAPTOR BRACKET

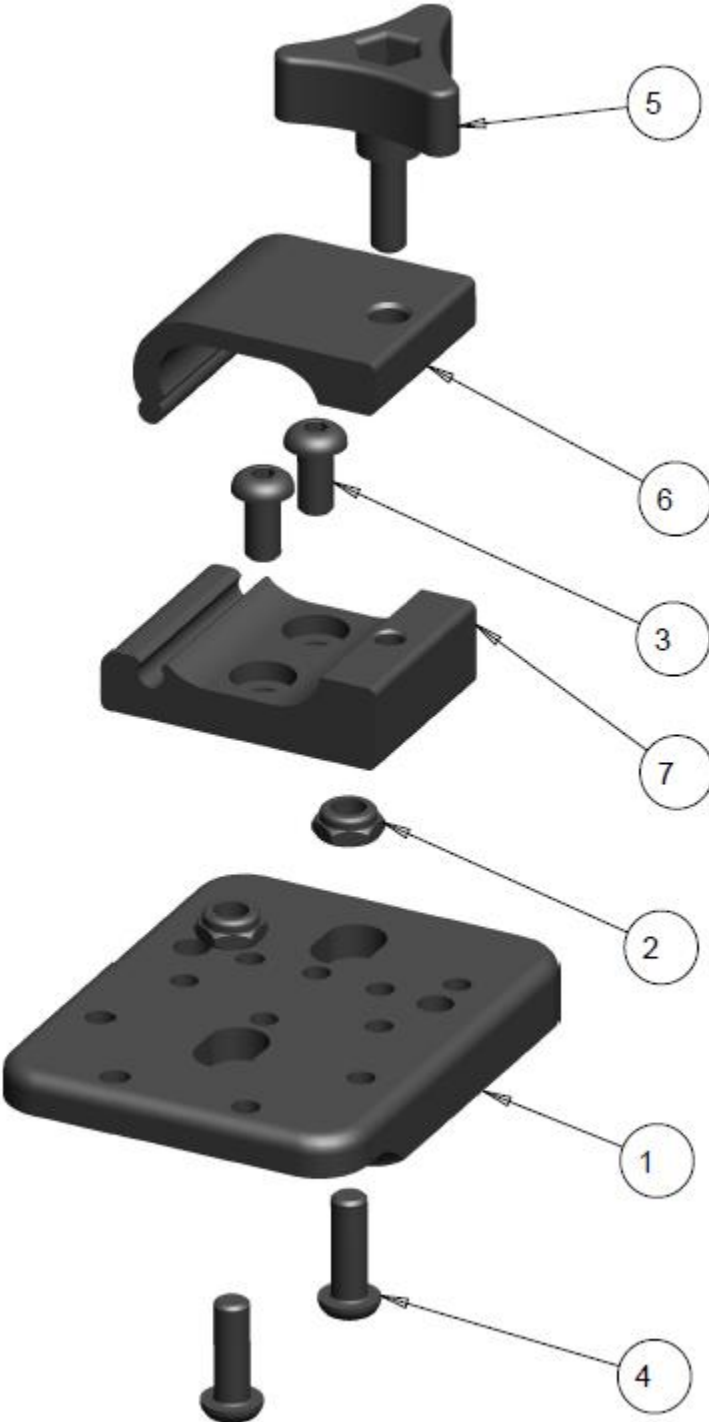


UNIVERSAL HEADREST BASE ADAPTOR BRACKET TABLE

ITEM	QTY	NOTES	PART NO.	DESCRIPTION	PRICE/ PER UNIT
1	1		1000218	UNIVERSAL MOUNTING PLATE, HEADREST	\$61.00
2	2		1000397	NUT, HEX-NYLOCK THIN, 1/4-20NC	\$2.00
3	2		1000411	SCREW,SOC BUTTON HEAD 1/4-20 X 0.500	\$3.00
4	2		1001391	SCREW, SOCKET HEAD BUTTON, 1/4-20NC X 3/4	\$3.00
5	1		1001464	HANDLE 3-POINT 1/4-20 X 3/4 STEM BLACK	\$8.00
6	1		1019474	OVAL UPPER CLAMP	\$16.00
7	1		1019475	OVAL TUBE LOWER CLAMP	\$20.00
	1		1003995	KIT, UNIVERSAL HEADREST ADAPTOR BRACKET (ITEMS 1-7)	\$120.00

NOTES:

HEADREST CURVED HINGED OVAL CLAMP

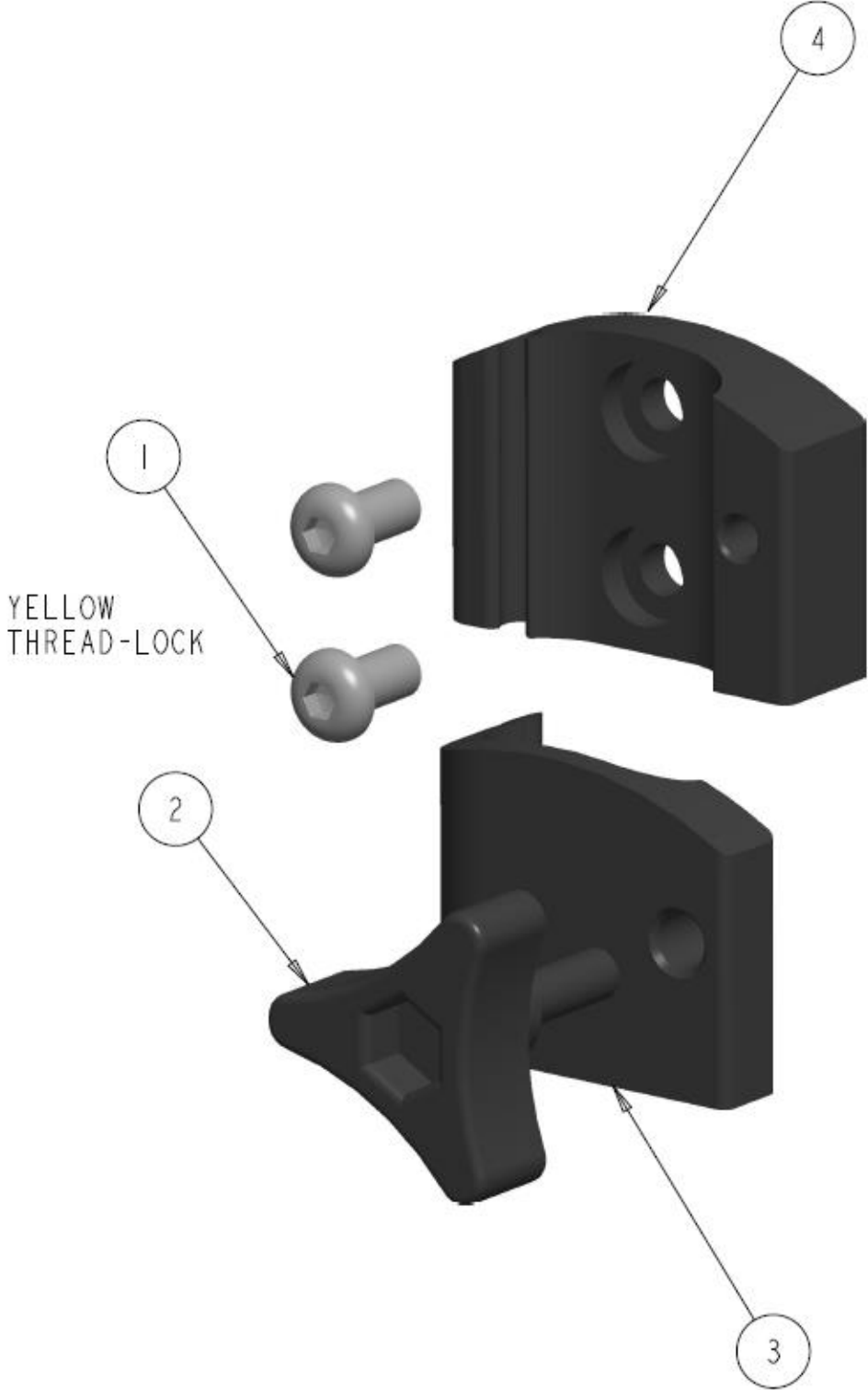


HEADREST CURVED HINGED OVAL CLAMP – TABLE

ITEM	QTY	NOTES	PART NO.	DESCRIPTION	PRICE/ PER UNIT
1	2		1000411	SCREW,SOC BUTTON HEAD 1/4-20 X 0.500	\$3.00
2	1		1001464	HANDLE 3-POINT 1/4-20 X 3/4 STEM BLACK	\$8.00
3	1		1019597	HINGED CURVED OVAL UPPER CLAMP	\$24.00
4	1		1019598	CURVED OVAL TUBE LOWER CLAMP	\$28.00
	1		1037667	KIT, COMPLETE HEADREST CURVED HINGED OVAL CLAMP	\$60.00

NOTES:

HEADREST CIRCULAR HINGED OVAL CLAMP

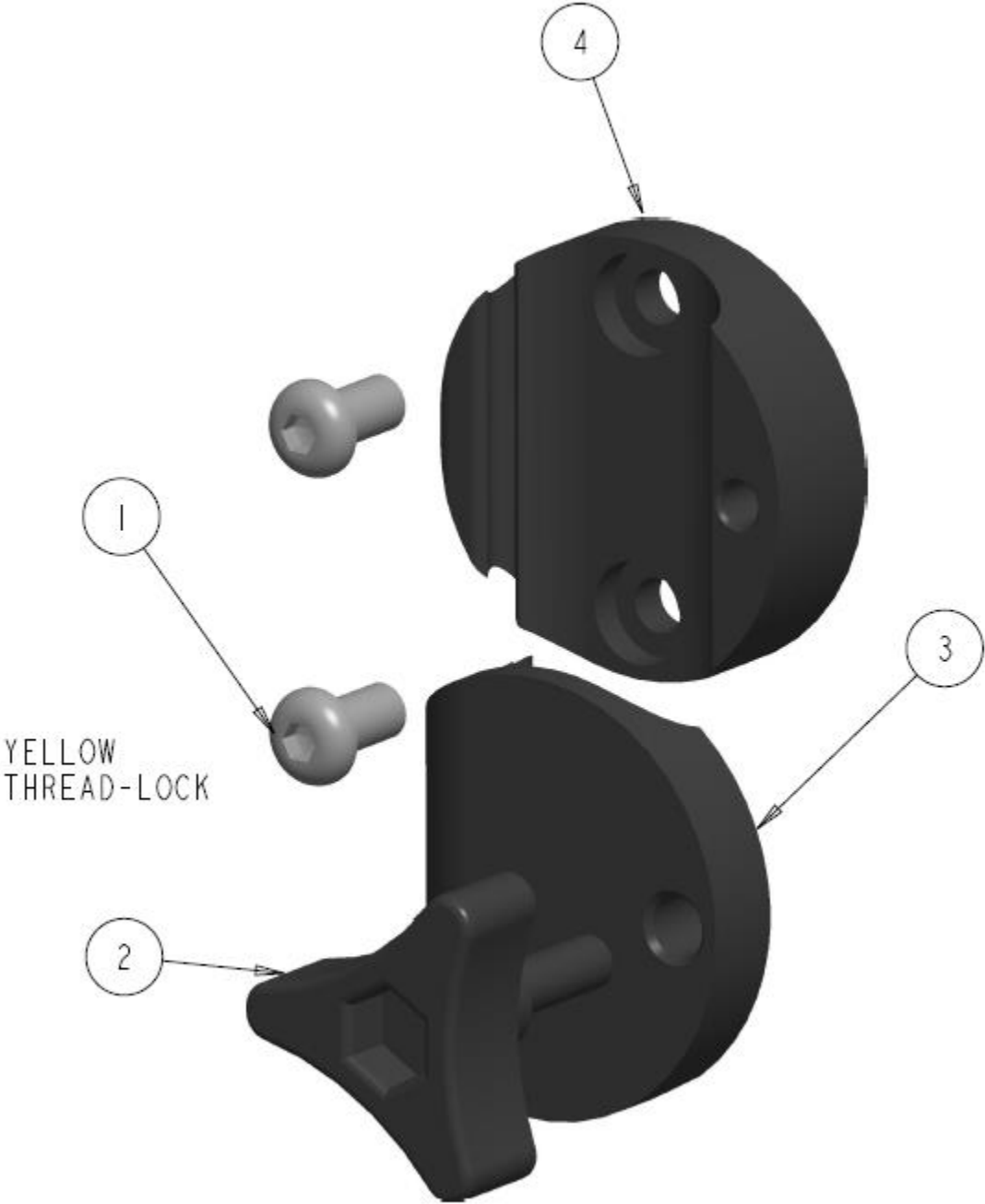

HEADREST CIRCULAR HINGED OVAL CLAMP - TABLE

ITEM	QTY	NOTES	PART NO.	DESCRIPTION	PRICE/ PER UNIT
1	2		1000411	SCREW,SOC BUTTON HEAD 1/4-20 X 0.500	\$3.00
2	1		1001464	HANDLE 3-POINT 1/4-20 X 3/4 STEM BLACK	\$8.00
3	1		1020988	HINGED CIRCULAR UPPER CLAMP	\$32.00
4	1		1020989	HINGED CIRCULAR LOWER CLAMP	\$36.00
	1		1037668	KIT, COMPLETE HEADREST CIRCULAR HINGED OVAL CLAMP	\$70.00

NOTES: