

**FUTURE
MOBILITY**



SEATING & POSITIONING
SOLUTION
Manufacturer: Future Mobility

USER MANUAL

Overbed Table

Model/Modèle: OT786



SPECS

BASE	19.5"W x 28"L
COLOUR	Base is white, support column is powder-coated, table top is walnut wood-grain
HEIGHT ADJUSTMENT	Approx. 30" - 45"
PRODUCT WEIGHT	30 lbs
PRODUCT WEIGHT CAPACITY	40 lbs.
WARRANTY	1 year limited

FEATURES

- Light upward touch required for lever and activating the spring-loaded mechanism allowing the table to elevate freely
- Unique H-base design permits use with chairs
- Our hospital grade rigid thermofoil top provides durability and the seamless spill guard edge to reduce infection control issues.
- Durable, heavy duty bases, painted welded tubular steel construction
- Supports up to 40 pounds of evenly distributed weight

CANADA

6750 Professional Court,
Mississauga, Ontario, L4V 1X6
1-888-737-4011

www.futuremobility.ca



USA

1 Buffalo River Place,
Buffalo, NY, 14210
716-783-9130

www.futuremobility.com



EASY INSTALL

★ ★ ★

2 EASY STEPS

★ ★ ★

EASY INSTALL

INSTRUCTIONS

TABLE PARTS GUIDE GUIDE DES PIÈCES DE TABLE



i TOOLS REQUIRED: 7/16 and 9/16 Wrench, 5/32 Allen key

1 ATTACH BASE TO POST



Insert the bolts and plate combination through the base to attach post. Use 9/16 wrench to tighten.

2 ATTACH TABLE TO THE POST



1

Slide the table top into the post (1). Tighten with 7/16 wrench. Reassemble bolt, nut and ratchet lever to post (2). Remove safety bolt (3).



2



3

OPERATING INSTRUCTIONS



(1) Shows the overbed table fully assembled. The table height can be adjusted by squeezing the ratchet lever towards the table top (2). Using your other hand support the other end of the table top while adjusting the height.